LipoNasal®

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DE

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Areas of application:

LipoNasal® Heuschnupfen serves to mitigate and concomitantly treat symptoms of hay fever (seasonal allergic rhinitis) as well as to humidify and care for the nasal mucosa. It is also suited for perennial allergic rhinitis (e.g. caused by animal allergens, mold and house dust mites).

LipoNasal® Heuschnupfen supports and promotes the natural healing process in irritation and inflammation of the nasal mucosa.

Directions for use:

Spray LipoNasal® Heuschnupfen 1 – 2 times into each nostril as needed. The treatment may be repeated several times a day. For more severe cases, LipoNasal® Heuschnupfen may be used more often.

Tips for correct use:

- Clean the inside of the nose before using the spray
 - Bend your head forward at an angle of approx. 45° („Looking at your own feet while standing“)
 - Hold the nasal spray upright. Place the head of the spray into one nostril and direct it slightly to the side of the nose (do not spray onto the nasal septum)
 - Inhale slowly through the nose and deliver a complete jet spray
 - Breathe out through the mouth
 - Repeat the same procedure on the other side of the nose.
- Please wipe off the head of the spray after each application and replace the protective cap after use.

In order to avoid possible infections, the spray bottle should only be used by the same person.

Contraindications:

LipoNasal® Heuschnupfen should not be used in the presence of hypersensitivity to one of the ingredients.

Notes:

The bottle is filled generously, because for technical reasons it cannot be completely emptied. It is guaranteed that the indicated quantity can be used.

Packaging contents, appearance and fragrance:

Due to its liposomal formulation, LipoNasal® Heuschnupfen is an opalescent liquid with a milky appearance and the characteristic natural fragrance of lecithin.

A brown-coloured opacity or residue may develop during storage, particularly in the area of the spring inside the pump unit. This is a characteristic of the vegetable component (soya lecithin), which has no effect on the effectiveness or tolerance, however.

Shelf-life and storage:

Store in a cool and dry place.

Keep out of reach of children.

Can be used up to 6 months after first use.

Do not use after expiry date.

When using for the first time, you may have to pump 3 or 4 times to fill the tube.

Date of last revision: 2018-02

Symbol/Symbol/Simbolo/Symbol:

| | | | | |
|--|---|--|---|--|
| | Hersteller | Fabriqué par | Prodotto da | Manufacturer |
| | Verfalldatum (JJJJ-MM: Jahr-Monat) | Date de péremption (AAAA-MM: Année-Mois) | Data di scadenza (AAAA-MM: Anno-Mese) | Expiry date (YYYY-MM: Year-Month) |
| | LOT-Nummer | Numéro de lot | Lotto numero | LOT-number |
| | Lagertemperatur 2°C – 25°C | Température de stockage 2°C – 25°C | Temperatura di conservazione 2°C – 25°C | Storage temperature 2°C – 25°C |
| | Gebrauchsinformation beachten | Observer la notice explicative | Leggere attentamente il foglio illustrativo | Observe instruction leaflet |
| | Nicht verwenden wenn die Flasche beschädigt ist | Ne pas utiliser si le flacon est endommagé | Non usare se la bottiglia è danneggiato | Do not use if the bottle is damaged |
| | Nach ersten Gebrauch 6 Monate verwendbar | A utiliser dans les 6 mois après la première utilisation | Può essere usato fino a 6 mesi dopo il primo utilizzo | Can be used up to 6 months after first use |